
# *SMART* *Pulser*

Quickstart Guide  
Schnellstartanleitung  
Guida Rapida  
Guide de démarrage rapide  
快速入门指南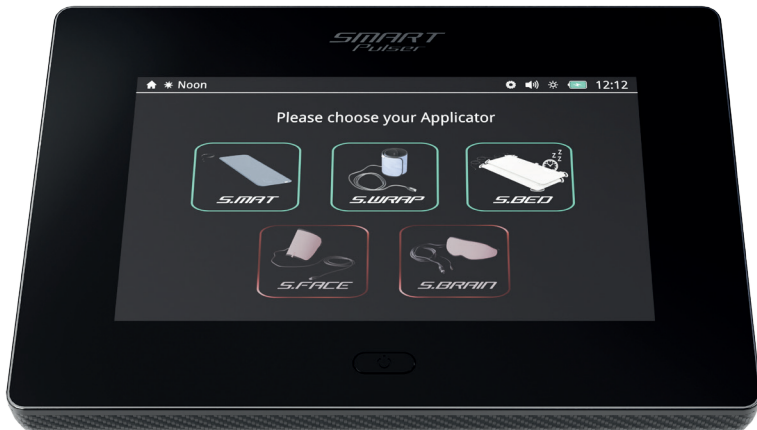

This SmartPulser QuickStart Guide provides a condensed overview of how to set up and begin using your SmartPulser-System. For detailed information, please download the full Operating Manual:

[www.smartpulser.com/manuals](http://www.smartpulser.com/manuals)

## 1. Package Contents

Ensure your system includes the following components (*content varies depending on your individual order*):

**SmartPulser Control Unit with pre-installed firmware, Power Supply and USB-Stick** (*for future software updates*)

**Applicators (e.g., S.Mat, S.Bed, S.Wrap)** (*depending on your individual order*)

**Quick Start Guide**

## 2. Setup & Installation

- Place your applicator (e.g., S.Mat) on a stable, even surface such as a floor, couch or lounger.
- Ensure the area is free of electro-smog. Keep devices like TVs, microwaves, and radio-based phones at least 1–2 meters away during use.
- Plug your chosen applicator into one of the sockets on the back of the control unit. Ensure the logo on the plug points upwards during insertion.

## 3. Powering On & First-Time Activation

**Power On:** Press and hold the on/off button on top of the device for several seconds until the start screen appears (*SmartPulser loading...*).

**Initial Configuration:** On first startup, set your preferred language and the system time. This is critical for the system's built-in bio-rhythm clock.

**Safety Confirmation:** Read the displayed legal contraindications (e.g., Epilepsy, Pregnancy, Electronic Implants) and press **CONTINUE**.

## 4. Choosing Your Mode

**After activation, you can select between two modes:**

**“Simply”:** Best for basic, easy-to-use functions.

**“Individually”:** Provides access to the full range of intensity and duration settings and advanced accessories like S.Brain or S.Face.

## 5. Starting a Session (“Simply” Mode)

1. Select your connected **applicator** icon from the home screen.
- 2.a **S.Mat and S.Bed:** Choose one of the five preset **Fast Start Programs**:
  - **Sleep:** Duration: 24 min., decreasing intensity at low levels.
  - **Relaxation:** Duration: 24 min., decreasing intensity.
  - **Performance:** Duration: 12 min., higher intensity.
  - **Activation:** Duration: 12 min., increasing intensity.
  - **Revitalization:** Duration: 24 min., low intensity.

- 2.b **S.Wrap:** Select the intensity level “Low,” “Medium,” or “High”.
3. Press **START** to begin the session.

## 6. Starting a Session (“Individually” Mode)

1. Select your connected **applicator** icon from the home screen.
- 2.a Choose the desired **intensity level:**
  - **sens, 10, 25** (= lower settings level)
  - **50, 100** (= medium settings level)
  - **200, 400** (=higher settings level)

followed by the duration of the application (usually between 8 and 24 minutes for whole body applications and 8-60 minutes for local applications (e.g.,S.Wrap).

- 2.b Or choose one of the **Fast Start Programs** (see 5.).
3. Press **START** to begin the session.

## 7. Session Controls

**Pause:** Use the PAUSE button to interrupt and resume the session.

**Stop:** Use the STOP button to end the session immediately.

### Important Safety Reminders

**Temperature:** Only operate the system in environments between +10 and +40 degrees Celsius (50-104 degrees Fahrenheit).

**Dizziness:** If you feel dizzy, stand up very slowly after a session.

**Cleaning:** Use a dry micro-fiber cloth for the touchscreen. Avoid getting water inside the housing.

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