

PEMF and circulation

Author:
Dr. med. Lukas Barwitz



The improvement of the blood circulation is one of the most frequently mentioned positive effects of PEMF application and thus application in numerous illnesses and sensory disturbances. In particular, from the improvement of the microcirculation is always talked about. We want to take a closer look at what actually microcirculation means and by what mechanisms pulsed electromagnetic fields (PEMFs) can help improve circulation.


What is microcirculation?

The term microcirculation refers to blood flow in the smallest vessels. These are composed of arterioles, capillaries and postcapillary venules. Also included are the terminal lymphatics.

Why is microcirculation so important?

In this so-called terminal flow path, the capillary network, the blood circulation performs its main task: the exchange of substances between blood and tissue. In other words, if the microcirculation does not work properly, this transfer in the tissue does not work properly and thus both the supply of oxygen and nutrients, as well as the removal of carbon dioxide and metabolic end products.

It is easy to see which negative health causes a malfunctioning (micro) circulation can have.

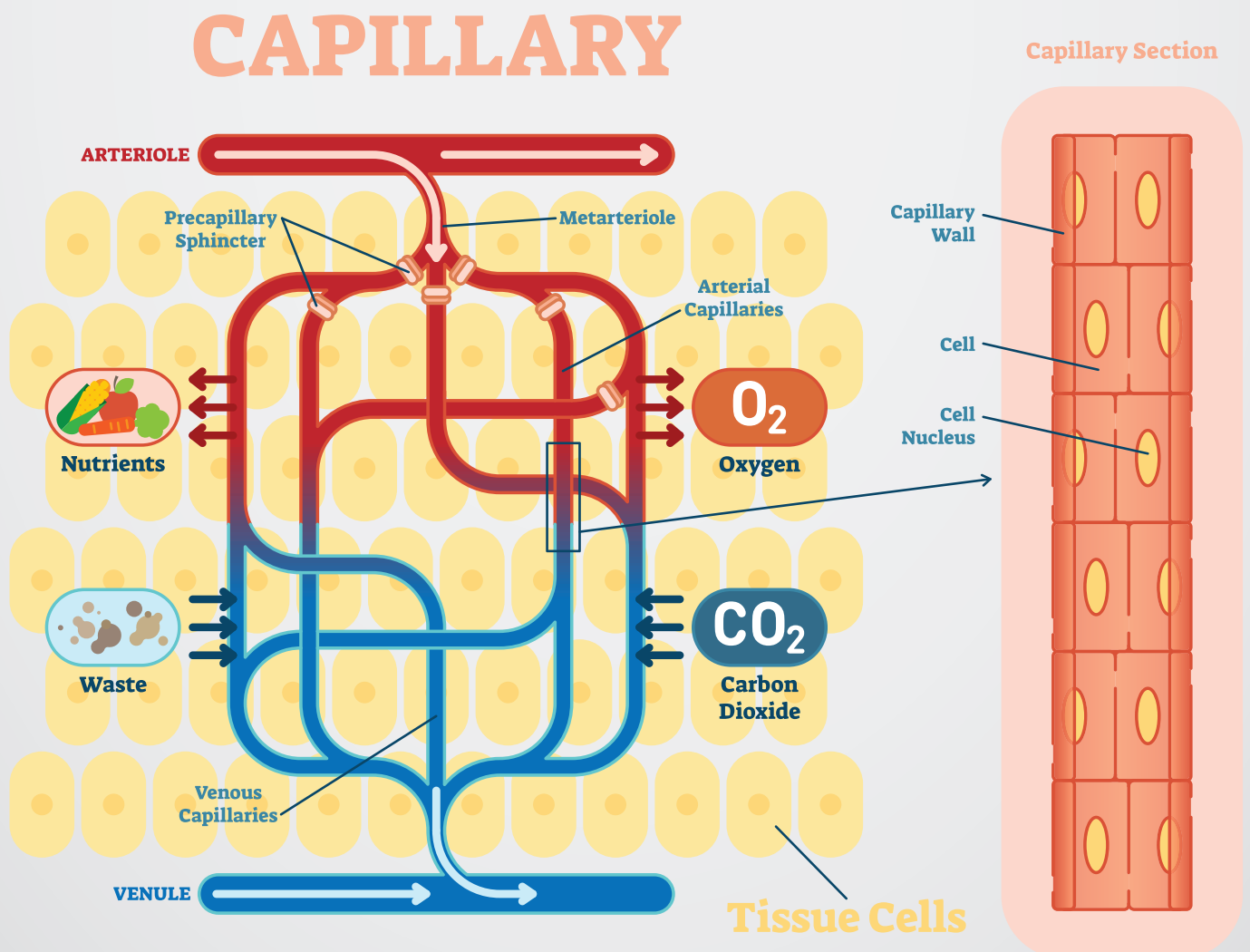

How can PEMFs influence circulation?

The positive effects of PEMFs on circulation can be explained by several mechanisms:

- Dilation of blood vessels (NO mediated vasodilation)
- Formation of new blood vessels through increased secretion of growth hormones FGF-2 / VEGF (neoangiogenesis)
- Reduction of edema and swelling
- Reduction of adhesion of platelets
- Reduction of fibrinogen and improvement of fibrinolysis
- Suggestions for improving the ductility of the red blood cells so that they can be transported through capillaries more easily
- Reduction of inflammation

Blood viscosity

Viscosity is a measure of the “thickness” of liquids. The higher the viscosity, the “thicker” is the liquid. A high viscosity thus characterizes a fluid as less fluid. The particles within a viscous fluid are more bonded together and consequently relatively immobile. The blood does not behave like a Newtonian fluid, but exhibits a non- proportional and erratic viscosity and is sometimes determined by the so-called Fåhræus-Lindqvist effect.

With the expression of the Fåhræus-Lindqvist effect, medicine refers to the characteristic behavior of the blood, whose viscosity changes depending on the diameter of the vessel. In small-diameter vessels, the blood is therefore less viscous, to prevent capillary stasis (congestion). In addition to the Fåhræus Lindquist effect, many other parameters determined the blood viscosity.

The viscosity of human blood depends, for example, on:

- Hematocrit (the proportion of all cellular components in the volume of blood)
- Flexibility of erythrocytes (red blood cells)
- Erythrocyte aggregation (juxtaposition of erythrocytes)
- Plasma viscosity
- Temperature
- Flow velocity

Since there is a close relationship between viscosity, flow dynamics, and the supply of nutrient and oxygen to body tissues, blood viscosity disorders can have serious consequences on the whole organism.

Circulation

Vasodilation is the dilation of the blood vessels. An abnormal or decreased production of nitric oxide (NO), as occurs in various diseases, affects blood flow and other vascular functions. In a healthy cell environment, blood vessels are constantly producing nitric oxide. In the inner lining of the blood vessels (endothelium), nitric oxide serves as a messenger to signal that the adjacent smooth muscle relaxes in the blood vessel wall, which dilates the blood vessels. The subsequent increase in blood flow leads to a decrease in both blood pressure and heart rate. Improvement of blood flow is considered to be one of the most important mechanisms for the health-promoting effect of PEMFs, including: Improve nutrient transport, repair molecules, stimulate growth factors, increase oxygen, and eliminate waste products¹.

Red blood cells

Red blood cells (erythrocytes) transport oxygen into tissue. The number of red blood cells we carry and produce is amazing. We produce more than 200 billion red blood cells per day, which is equivalent to two million per second and they make up 70% of all cells in our body. Red blood cells are different from other cells in that they lack a nucleus, which creates more space for hemoglobin. Because they do not contain mitochondria, they do not use the oxygen that they carry. This also means that they contain no DNA and are therefore unable to repair or regenerate much. Erythrocytes form in the bone marrow and have a lifespan of about 110-120 days before being degraded and absorbed by the body. Through all the underlying mechanisms involved, PEMF therapy may favor the formation, function and absorption of red blood cells.

Like all membranes, the membranes of erythrocytes consist of lipids and proteins that allow maximum flexibility as the cells move through the circulatory system. Most of the oxygen transfer takes place in the microcirculation system, which mainly consists of capillaries. Capillaries are so incredibly small (5 to 10 μm in diameter) that the red blood cells (7.5 μm in diameter) must be deformable to push through.

A phenomenon called the “rouleaux effect” can make this process difficult or impossible. A Rouleaux formation consists of a stack of red blood cells that stick together. It resembles a stack of coins and is therefore commonly referred to as “roll forming”. Due to the disc-like shape of erythrocytes, they are somewhat predisposed to this type of clumping. Capillaries can each pass only a single red blood cell. Therefore, it is necessary for healthy blood circulation

¹ McKay JC, Prato FS, et al. A literature review: the effects of magnetic field exposure on blood flow and blood vessels in the microvasculature. *Bioelectromagnetics*. 2007 Feb;28(2):81-98.

to prevent a “rouleaux effect” as much as possible. Because PEMF therapy enables balanced cell membrane charge, it has a direct impact on this effect. Properly charged red blood cells repel each other. Apart from preventing the “rouleaux effect”, this separation of red blood cells allows for a larger available surface area for oxygen and nutrient uptake and exchange. In addition, magnetic fields appear to enhance the release of oxygen from hemoglobin, with exposure of only ten to thirty minutes increasing the rate of release of oxygen for several minutes to several hours².

Anticoagulant effects

PEMF therapy addresses many of the mechanisms that lead to increased blood viscosity and blood clotting, including the reduction of platelet adhesiveness, the reduction of fibrinogen and the improvement in fibrinolysis and the enhancement of red blood cell deformability (making them easier to move through capillaries). These anticoagulant effects seem to be the case for virtually all types of PEMFs³.

Anti-edema activity

The edema is a swelling caused by excess fluid in the tissue. Edema tissues get rid of oxygen, nutrients and circulation. PEMF therapy has a positive effect on swelling and edema by improving cell metabolism by acting directly on the sodium-potassium pump in the cell membrane. Any type of cell or tissue damage causes edema due to leakage of fluids from the cells and blood vessels. Improving the circulation helps to remove this excess fluid and prevent further accumulation of fluid. Anti-edema effects are rapid after using PEMFs³.

PEMFs not only improves blood circulation but also has a positive effect on lymphatic vessels, stimulates lymphatic drainage and reduces the production of lymphoedema, thus stimulating the immune system and correcting the edema³.

Anti-inflammatory reaction

Inflammation is the body’s defense against cell injury, regardless of the cause. The goal of acute inflammation is to contain and eliminate the irritant and initiate the repair process. There are three stages to the acute inflammatory response: vasodilation (enhanced by PEMFs), increased vascular permeability (enhanced by PEMFs by promoting action potential and ion flow), and white blood cell transmission and accumulation at the site of injury.

² Muehsam D, Lalezari P, et al. Non-thermal radio-frequency and static magnetic fields increase rate of hemoglobin deoxygenation in a cell-free preparation. PLOS One. 2013 Apr 12;8(4):e61752.

³ Jerabek J, Pawluk W. Magnetic therapy in Eastern Europe: a review of 30 years of research. Publ Advanced Magnetic Research of the Delaware Valley, Chicago, 1996.

A chronically inflamed tissue is like a swamp - there is only little fluid flow and therefore little drainage. By increasing the blood flow through vasodilation, the environment of the swamp changes considerably. PEMF therapy also increases the production of growth factors that help combat inflammation.

Damaged cells have impaired electrical capabilities and therefore lack the action potential needed to perform basic cellular functions including oxygen uptake and sodium and potassium exchange. PEMF therapy restores the electrochemical gradient of the cell membrane, restoring normal electrical potential between cell membranes and thus normal cell function³.

Oxygen

Low oxygen concentrations cause cell damage, with the severity and duration of oxygen deprivation in tissues indicative of the degree of damage. The vast majority of cellular functions depend on adequate oxygenation. PEMFs can significantly increase oxygen levels in tissues by improving blood circulation, helping the body carry oxygen from the air into the lungs and into the blood, and positively affect the ability of hemoglobin to transport and deliver oxygen.

One study focused on the effect of magnetic fields on the gas transport function of blood during oxygen deprivation. They found that magnetic therapy changed the hemoglobin molecules from a form that is less reactive to oxygen into a form that is more reactive and thus positively influences the gas transport function.

Another study was performed on patients with terminal emphysema. These patients received PEMF therapy (30 minutes per day for seven days) in addition to their usual medical care. The addition of magnetic field therapy improved blood oxygen levels by up to 21%, and all patients reported subjectively better endurance. It is conceivable that these results would have been even more significant if the treatments were performed earlier in the disease and for a longer time⁴.

One of the earlier demonstrations of the benefits of PEMFs to improve oxygenation was made in the early 1990s in Germany with a large PEMF plate. They found that PEMF effects can be enhanced during periods of high muscle activity, alcohol intake, sleep, or after inhalation of CO₂. Hyperventilation and large meals would reduce the level of impact. In their studies, they found that the blood volume increased, the oxygen content was boosted and the pH became more alkaline⁵.

4 Pawluk W, MD, MSc, Layne C. Power Tools for Health: How pulsed magnetic fields (PEMFs) help you. FriesenPress Nov 2017. p. 42

5 Warmke U. Survey of some working mechanisms of pulsating electromagnetic fields PEMF). Bioelectrochem Bioenerg 27(3):317-320,1992.

Further measures to stimulate the blood circulation

- **Exercise:** Daily exercise such as walking or regular moderate endurance sports - if possible in the fresh air - not only promotes blood circulation, but also strengthens vessels and trains the muscle pump.
- **Garlic and ginger:** They are considered «tube-free» and prevent arteriosclerotic deposits on the vessel walls.
- **Quit smoking:** Smoking is known to be a risk factor for almost all maladies - also due to its negative influence on the circulation.
- **Kneipp cures or alternating showers:** gentle vascular training according to Pastor Kneipp, such as treading water.
- **Proper diet:** A Mediterranean-style diet with plenty of vegetables, fruits whole grains and raw foods help with weight loss and has a positive effect on blood lipid levels. If possible, fish should be on the menu at least twice a week. On the other hand, meat and sausage have an unfavorable effect on the blood circulation and should only be eaten in moderation.
- **Drink enough:** Blood is 90% water. Inadequate water intake also means inadequate blood supply and may cause dizziness or headache.
- **Reduce Obesity:** Obesity increases the risk of high blood pressure, high cholesterol and diabetes, thereby favoring arteriosclerosis.

Lukas Barwitz, M.D.

Medical Disclaimer

All information and research published by the Swiss Bionic Solutions GmbH is for informational purposes only. They have been carefully researched and reviewed by the authors, however no responsibility for their correctness can be taken. Readers are encouraged to confirm the information contained herein with other sources. Patients and consumers should review the information carefully with their professional health care provider. The information is not intended to replace medical advice offered by physicians. A guarantee of treatment success cannot be accepted by the publisher and the authors under any circumstances. The Swiss Bionic Solutions GmbH will not be liable for any direct, indirect, consequential, special, exemplary, or other damages arising therefrom.



Swiss Bionic Solutions Schweiz GmbH

Schulhausstrasse 17 | 8834 Schindellegi, Schweiz
Telefon: +41 (62) 295 5951 | Fax: +41 (62) 295 5952 | E-Mail: ch@swissbionic.com

Swiss Bionic Solutions Deutschland GmbH

Biberacher Str. 87 | 88339 Bad Waldsee, Deutschland
Telefon: +49 (7524) 996 950 | Fax: +49 (7524) 996 9518 | E-Mail: de@swissbionic.com

Swiss Bionic Solutions USA Inc.

12330 SW 53rd Street | Suite 703 & 704 | Cooper City | Florida 33330, USA
Telefon: +1 (954) 766 4153 | Fax: +1 (954) 766 4156 | E-Mail: us@swissbionic.com

Swiss Bionic Solutions Canada Inc.

1195 North Service Rd W. Unit B8 | Oakville, ON, L6M 2W2, Canada
Telefon: +1 (905) 465 0753 | Fax: +1 (1 866) 792 8182 | E-Mail: ca@swissbionic.com

Swiss Bionic Solutions Asia Ltd.

998 Canton Road | Mongkok | Kowloon | Hong Kong
Telefon: +852 2337-8774 | E-Mail: asia@swissbionic.com

